



BIOLABO REAGENTS
www.biolabo.fr
MANUFACTURER:
BIOLABO SAS,
 Les Hautes Rives
 02160, Maizy, France

COATROL 2 Level 2

For Internal Quality Control in haemostasis

REF 13972 : 6 x 1 mL



Made In France

TECHNICAL SUPPORT AND ORDERS

Tel: (33) 03 23 25 15 50

Fax : (33) 03 23 256 256

support@biolabo.fr

Latest revision: www.biolabo.fr

I: corresponds to significant modifications

INTENDED USE

| This control plasma is designated for professional use in laboratory (manual or automated method).

It is used to monitor the reproducibility and accuracy of following analysis performed with BIOLABO Reagents:

REF 13702, 13704, 13712: BIO-TP LI

REF 13885, 13880 and 13881: BIO – TP

REF 13560 and 13570 BIO-CK

REF 13660 and 13670 BIO-SIL

REF 13450 and 13451 BIO-FIBRI

REF 13302: Factor II

REF 13305: Factor V

REF 13307: Factor VII

REF 13308: Factor VIII

REF 13309: Factor IX

REF 13310: Factor X

REF 13311: Factor XI

REF 13311: Factor XII

REAGENTS

REF 13972 COATROL 2 Level 2



Freeze-dried, human citrated plasmas

Human origin

SAFETY CAUTIONS (1) (2)

- Material Safety Data Sheet is available upon request.
- Each donor unit used to manufacture this product was tested and found non-reactive for HbsAg, antibody to Hepatitis C and antibody to HIV-1/HIV-2.
- However, no test method can offer complete assurance that infectious agents are absent. All specimens or reagents from biological origin should be handled as potentially infectious, in accordance with good laboratory practices using appropriate precautions.
- Waste disposal: Respect legislation in force in the country.

| Any serious incident that has occurred in connection with the device is notified to the manufacturer and the competent authority of the Member State in which the user and/or patient is based.

MATERIAL REQUIRED BUT NOT PROVIDED

1. Basic medical analysis laboratory equipment
2. Control Plasmas REF 13971
3. Reference plasma REF 13970 or TP CAL SET REF 13965

REAGENTS PREPARATION

- Open the vial carefully and add exactly the volume of demineralised water **stated on the label**
 - Wait for 15 minutes at room temperature.
 - Gently agitate before use (avoid the formation of foam).
- WARNING: DO NOT SHAKE. STORE AWAY FROM LIGHT.**

STABILITY AND STORAGE

Stored away from light, well capped in the original vial at 2-8°C, when stored and used as described, reagents are stable:

Unopened:

- Until expiry date stated on the label of the kit.

Once opened:

- R1 must be reconstituted immediately,

| Once reconstituted, plasma is stable:

- 10 hours at 2-25°C
- 5 days at -20°C

Don't use reconstituted plasma after expiry date stated on the label

PROCEDURE

Run in accordance with the IFU of the reagent used § **CONTROL**

LIMITES

Factors which may interfere with the result are:

- Bacterial contamination.
- The volume measured to reconstitute the plasma.
- The setting of the instrument.
- Temperatures

REFERENCES

- (1) *Occupational Safety and Health Standards; Bloodborne pathogens (29CFR1910.1030) Federal Register July 1, (1998) ; 6, p.267-280*
- (2) *Directive du conseil de l'Europe (90/679/CEE) J. O. de la communauté européenne n°L374 du 31.12.1990, p.1-12*
- (3) *Section 5.6 of ISO 17511- Measurements of quantities in biological samples- metrological traceability of values assigned to calibrators and controls*

Manufacturer	Expiry date	In vitro diagnostic	Storage temperature	Dematerialized water	Biological risk
REF		LOT			→
Product Reference	See Insert	Batch number	Store away from light	Sufficient for	Dilute with

VALUES AND RANGES (3)

Make sure that the batch number stated on the label of the vial corresponds to the batch number indicated here above.

Fibrinogen and Factors II, V, VII, VIII, IX, X, XI are determined using secondary Standards traceable to International Standards as follows:
SSC/ISTH Secondary Coagulation Standard Lot #4 NIBSC code: SSCLOT4

LOT _____	Semi-automated and full-automated methods, including: BIOLABO SOLEA 100, BEHNK THROMBOLYZER Series	
	Assigned Values	Range
BIO - TP LI: INR (Low ISI) (%) Prothrombin level or PT		
BIO - TP: INR (High ISI) (%) Prothrombin level or PT		
BIO - CK: Activated partial thromboplastin time (sec) or APTT (Kaolin)		
BIO - SIL: Activated partial thromboplastin time (sec) or APTT (Silica)		
BIO-FIBRI : Fibrinogen (mg/dL)		
Factor II- Deficient plasma (%)		
Factor V - Deficient plasma (%)		
Factor VII - Deficient plasma (%)		
Factor VIII - Deficient plasma (%)		
Factor IX - Deficient plasma (%)		
Factor X - Deficient plasma (%)		
Factor XI - Deficient plasma (%)		
Factor XII - Deficient plasma (%)		