



**BIOLABO**  
www.biolabo.fr

**MANUFACTURER:**  
**BIOLABO SAS,**  
Les Hautes Rives  
02160, Maizy, France

# MICROALBUMIN Standard Set

For calibration of quantitative determination of the excretion of albumin (MAL) in the human urine by Turbidimetric Immunoassay

REF 23013 R1 1 x 1 mL R3 1 x 1 mL R5 1 x 1 mL  
R2 1 x 1 mL R4 1 x 1 mL

## TECHNICAL SUPPORT AND ORDERS

Tel : (33) 03 23 25 15 50

Fax : (33) 03 23 256 256



IVD IN VITRO DIAGNOSTIC

## PRINCIPLE AND INTENDED USE

Ready to use, Standard Set for the preparation of reference curve for quantitative immunochemical determination of the excretion of albumin (MAL) in the human urine.

Suitable for manual procedure or automated instruments.

## REAGENTS

**MAL 1 to MAL 5**

5 vials of MAL Standards (5 different levels)

Liquid stabilized human plasmas (Sodium azide 0.95 g/L).

The concentration of each standard is batch-specific (see § ASSIGNED VALUES).

## SAFETY CAUTIONS (1) (2)

BIOLABO reagents are designated for professional, in vitro diagnostic use.

- Each individual donation intended for use in manufacture of protein standard serum was tested negative for hepatitis B surface antigen (HBsAg), anti-hepatitis C (anti-HCV) and anti-HIV 1 and HIV 2 by FDA approved test.
- However absence of infectious agents can never be proven, this plasma and all specimens should be handled as potentially infectious, in accordance with good laboratory practices using appropriate precautions. Respect legislation in force in the country.
- Use adequate protections (overall, gloves, glasses).
- Do not pipette by mouth.
- In the event of exposure the directive of the responsible health authorities should be followed.
- Reagents contain sodium azide (concentration < 0.1%) which may react with copper and lead plumbing. Flush with plenty of water when disposing.
- Material Safety Data Sheet is available upon request.

Waste disposal: Respect legislation in force in the country.

## REAGENTS PREPARATION

Ready for use

## MATERIAL REQUIRED BUT NOT PROVIDED

1. Basic medical analysis laboratory equipment.
2. Reagents and controls.

REF 23010, REF 23011	MICROALBUMIN Reagents
REF 23014	MICROALBUMIN, Control

## INTERFERENCES

Factors which may influence results are bacterial contamination, respect of automated instrument procedure, temperature control...

## STABILITY AND STORAGE

Store at 2 - 8° C, away from light in well capped original vial.

Do not freeze.

- Unopened, Standards are stable until expiry date stated on the label.
- Once opened, Standards are stable at least for 6 weeks when free from contamination.

## PROCEDURE

This Standard Set should be used with BIOLABO reagents REF 23010, REF 23011 or kits referring to the same method in accordance with technical data sheet of the reagent in use. Standards have to be handled as patient serum.

## QUALITY CONTROL

BIOLABO MAL Control REF 23014

or other assayed control plasmas referring to the same method.

It is recommended to :

- ✓ Participate to external quality control program.
- ✓ Control as indicated in technical sheet of the reagent.
- ✓ Validate Calibration values when using other reagents that BIOLABO reagents.

## ASSIGNED VALUES (3)

Make sure that the batch number stated on the label of each vial corresponds to the batch number indicated below.

			Albumin (mg/L)
MAL1	LOT	051431A1	19
MAL2	LOT	051431A2	53
MAL3	LOT	051431A3	106
MAL4	LOT	051431A4	201
MAL5	LOT	051431A5	376

Values of MAL Standards are traceable to Reference Material (RPPHS/CRM470) from IFCC.

## REFERENCES

- (1) Occupational Safety and Health Standards ; Bloodborne pathogens (29CFR1910.1030) Federal Register July 1, (1998) ; 6, p.267-280
  - (2) Directive du conseil de l'Europe (90/679/CEE) J. O. de la communauté européenne n°L374 du 31.12.1990,p.1-12
- TIETZ N.W. Text book of clinical chemistry, 3<sup>rd</sup> Ed. C.A. Burtis, E.R. Ashwood, W.B. Saunders (1999) p.520

